

StetPro Handipak

Instructions for use

- 1) Open lid by pressing upwards on lid tab at front of handipak (opposite side to the hinge)
- 2) Align your stethoscope head (diaphragm side) with centre of mouth of handipak and place it flat onto the membrane which is uppermost
- 3) Press stethoscope head perpendicularly downwards on the uppermost membrane to a depth of 0.5-1 cm, or until you hear an audible click which denotes the membrane being dispensed onto the stethoscope head. **NOTE:** The dispensing mechanism of StetPro is designed for light pressure. Heavy pressure may damage the dispensing mechanism
- 4) Withdraw stethoscope head out of dispenser
- 5) Use the stethoscope as normal
- 6) On completion of your examination dispose of membrane by applying pressure with your thumb on the rolled part of the membrane which is on the reverse side of the diaphragm, ensuring that you don't touch the surface of membrane that has been in contact with the patient
- 7) The next membrane is automatically ready or use

Trouble-shooting

- 1) Membrane does not rise to the top of dispenser. Try the following:
 - Gently tap handipak onto a flat hard surface, e.g. table top
 - Gently squeeze sides of tub
 - Press gently on edge of membrane inside the dispenser (this side of membrane does not come into contact with the patient)
 - Pinch the top membrane and pull it out of dispenser. This may release a block

- 2) As you press down on top membrane it does not dispense a membrane onto the stethoscope head:
 - This is probably due to the membrane not being fully at the top (housed), generally a second effort is all that is needed to dispense the membrane
 - If repeated efforts are not successful repeat the steps above